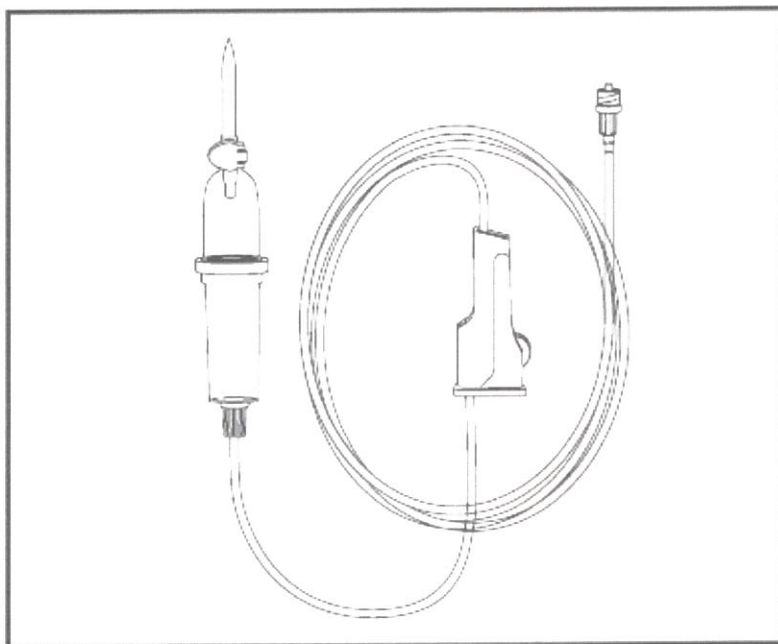
Intrafix® SafeSet

GB Instructions for use

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
B | BRAUN

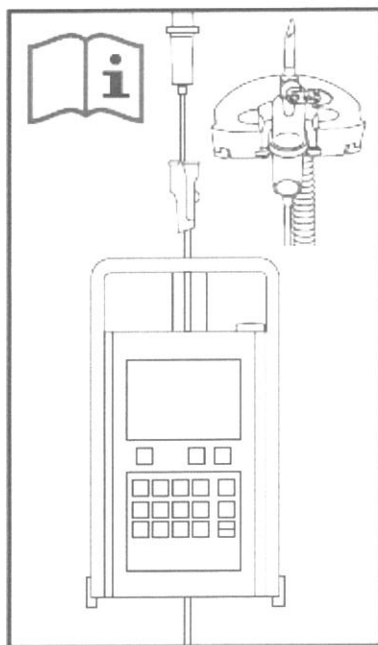P = Pressure
(Druck)

GB Development and design according to ISO 8536-8-IS-P

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

GB I.V. administration set. Pressure resistant (2 bar). Indications: to administer I.V. solutions/ medication [e.g. fluid/electrolyte balance, parenteral nutrition, volume expanders, antibiotics, cytostatics, other drugs etc.] by gravity and/or via a non-dedicated infusion pump using standard infusion lines made of PVC with ID 3 and OD 4,1 mm. Full list of suitable infusion pumps on request. Please refer to manufacturers instruction for use.

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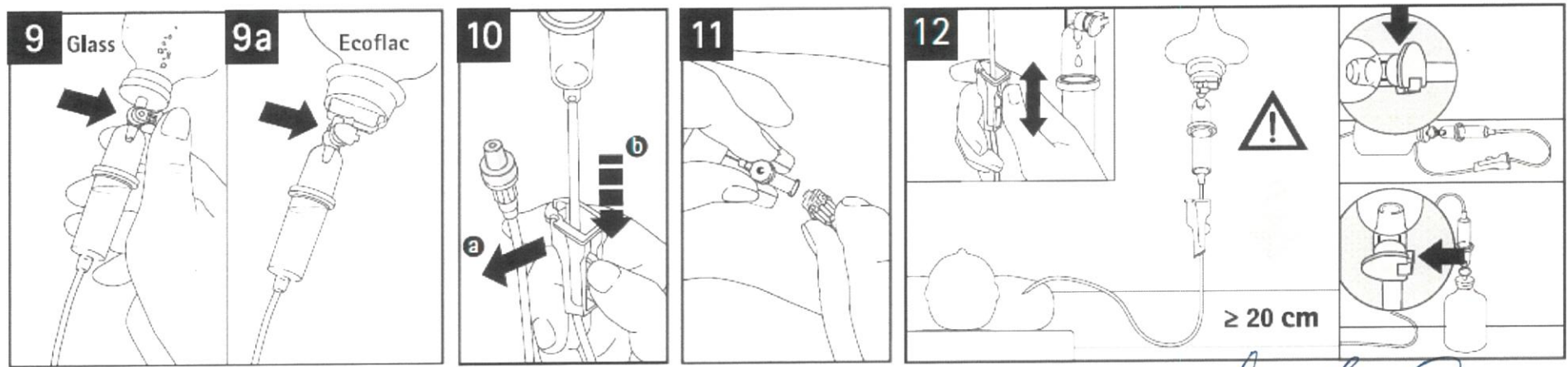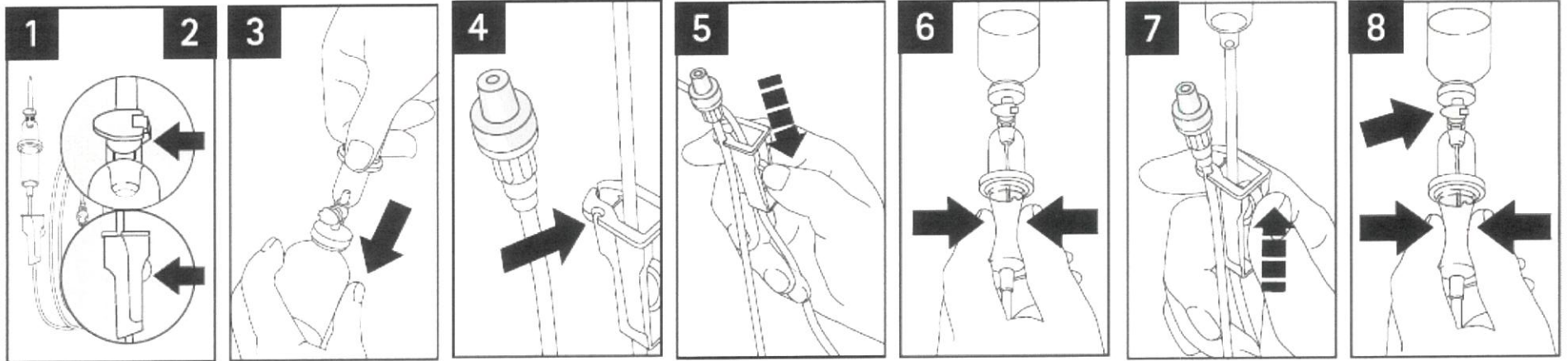
Intrafix® SafeSet

G = Gravity
(Schwerkraft)

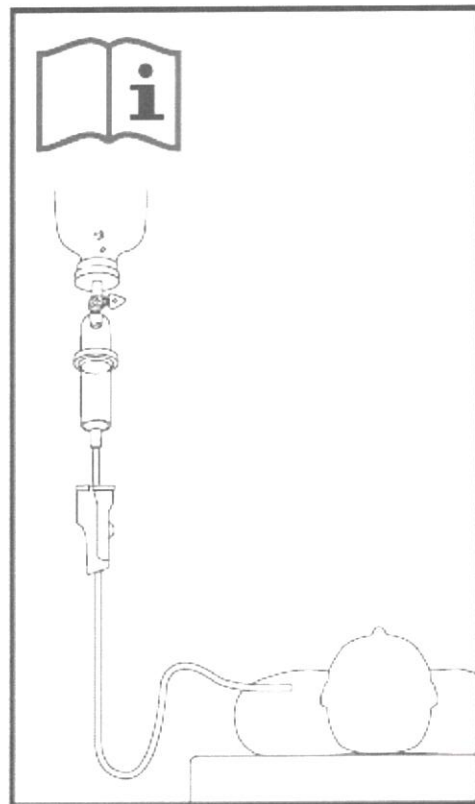GB Development and design according to ISO 8536-4-IS-G

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

GB I.V. administration set. For gravity infusions. Indications: to administer I.V. solutions/ medication [e.g. fluid/electrolyte balance, parenteral nutrition, volume expanders, antibiotics, cytostatics, other drugs etc.] by gravity.

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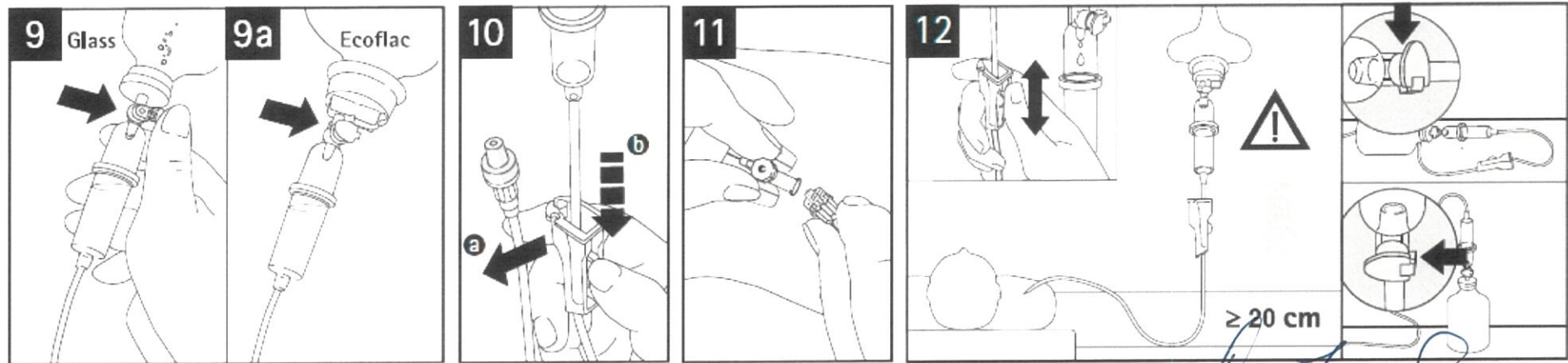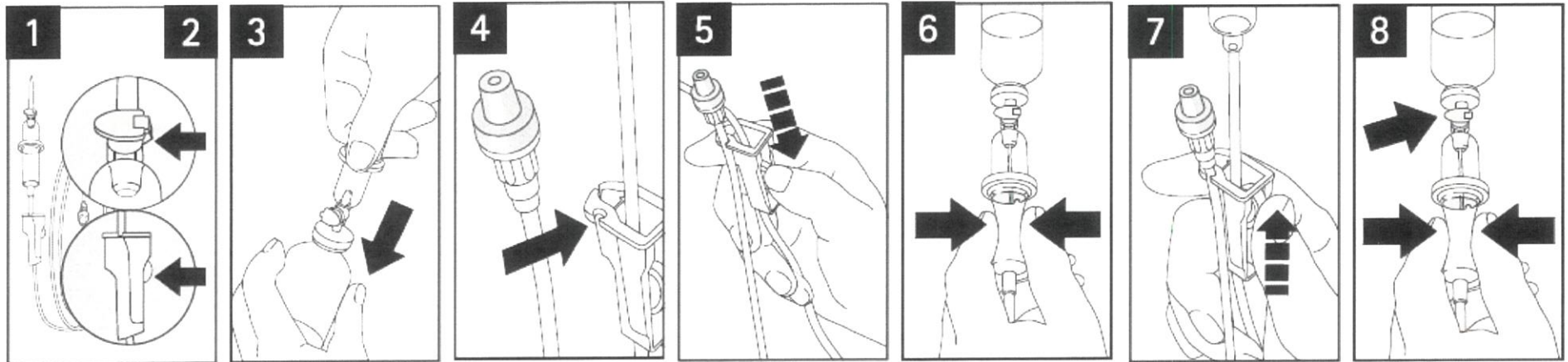K. J. J. J.
B. BRAUN B. Braun Melsungen AG
 34209 Melsungen
 Germany

Warning notices

GB Re-use of single-use devices creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.

Sterile, non-pyrogenic

Do not use if protective caps are missing or loose.

Caution! If a pneumatic cuff is used during infusion, remove air from container beforehand. Observe general safety precautions!

On bags and Ecoflac, leave vent flap closed.

Measured value of bolus volume in "ml/ 1 m" (tubing length) at 40 °C, 200 kPa (2 bar). To calculate the bolus volume of the individually placed i.v. administration set the declared value has to be multiplied by the length of the tubing (measured in "m") distal to the pump.

Intrafix® Primeline / Intrafix® Safe-Set has a pore size of $\leq 15 \mu\text{m}$. The infusion set is not suitable for infusion of blood and cell-containing blood products.

Also, the infusion of highly viscous solutions, lipid emulsions $> 10\%$ and certain suspensions can affect the flow rate.

With regard to this, please also follow the information in the relevant summary of product characteristics.

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