
3M


Littmann[®]
Brand

Master Classic II[™]
Stethoscope

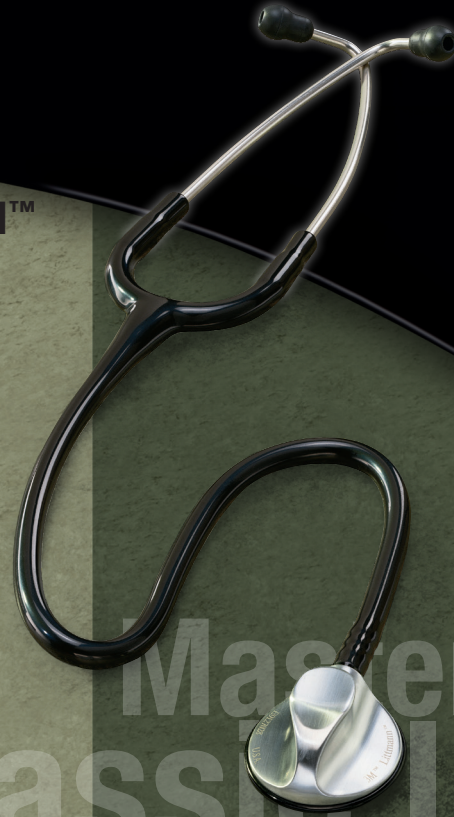Master
Classic II

Figures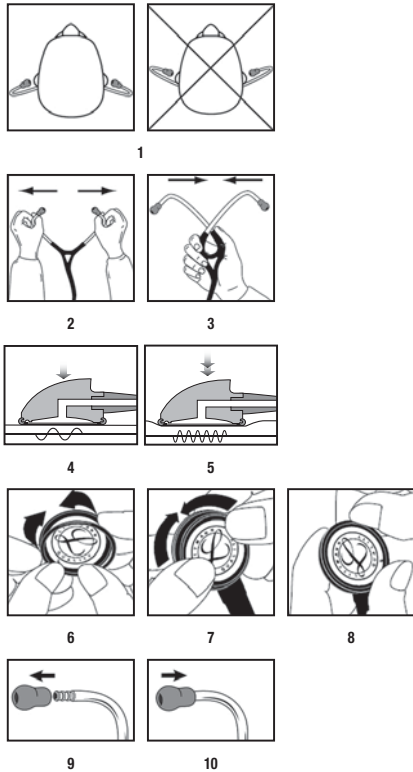

1

3M™ Littmann® Master Classic II™ Stethoscope User Manual

en

Intended Use: The 3M™ Littmann® Master Classic II™ Stethoscope is intended for use by healthcare professionals for medical diagnostic purposes only. It can be used for auscultation of heart, lung, and other body sounds.

Please report a serious incident occurring in relation to the device to 3M and the local competent authority (EU) or local regulatory authority.

Instructions for Use (see Figures pp. 1):

- **Place the Stethoscope in Your Ears:** The eartips should point forward toward your nose (Figure 1).
- **Adjust the Headset Tension for Comfort and Sound Quality:** Pull the eartubes apart (Figure 2) or squeeze them together (Figure 3).
- **Listen to Low and High Frequency Sounds with the Littmann Tunable Diaphragm:**
 - To emphasize lower frequency sounds, use light pressure against the patient (Figure 4).
 - To emphasize higher frequency sounds, use firm pressure against the patient (Figure 5).

Removing the Diaphragm:

Remove the two-piece rim/diaphragm assembly by rolling the rim off the edge of the chestpiece. Then remove the diaphragm from the rim.

Assembly of the Tunable Diaphragm:

Put the rim on the diaphragm by starting with the rim positioned above the legible side of the diaphragm and inserting the edge of the diaphragm into the rim (Figure 6). Then attach the rim/diaphragm assembly to the chestpiece by slowly rolling the rim around and over the chestpiece edge using both thumbs (Figure 7). Visually inspect the flexible edge of the diaphragm where it meets the rim and adjust as needed to achieve a smooth fit (Figure 8).

Cleaning, Disinfection and Storage:

Clean between exams using a 70% isopropyl alcohol wipe or disposable wipe with soap and water. A 2% bleach solution may be used to disinfect your stethoscope; however, the tubing may become discolored after exposure to bleach.

The tunable diaphragm and eartips may be removed for cleaning. Ensure all parts and surfaces are dry before reassembly. To remove eartips from the headset, pull firmly (Figure 9). To attach eartips, push the *small* side firmly onto the eartube until it snaps fully into place (Figure 10). To prevent ear canal injury, be sure eartips are firmly attached to the headset before inserting them into your ears.

To prevent stethoscope damage, do not: immerse your stethoscope in any liquid, subject your stethoscope to any sterilization process, or store in extreme heat. To prevent staining of stethoscope tubing, avoid contact with pens, markers, newsprint, or other printed material. It is good practice to wear your stethoscope over a collar whenever possible.

Disposal: Dispose of contents/container in accordance with the local/regional/national/international regulations.

Littmann Stethoscope Service and Warranty Program: The Littmann Master Classic II stethoscope is warranted against any defects in material and workmanship for a period of three (3) years. Within the warranty period, repairs will be made without charge upon the return of the stethoscope to 3M, except in cases of obvious abuse or accidental damage. For service and repair in the USA, please visit www.littmann.com/service or call 1-800-292-6298. If you are outside of the USA, please visit www.littmann.com for the contact information of your local 3M office.

For more information, visit www.littmann.com.

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3M, Littmann, the L Littmann logo, the L, and Master Classic II are trademarks of 3M. Used under license in Canada./3M, Littmann, le logo L Littmann, le L et Master Classic II sont des marques de commerce de 3M, utilisée sous licence au Canada.
© 2019, 3M. All rights reserved./Tous droits réservés.
**3M Deutschland GmbH**